Powering the Unit

1. Press **DOWN ARROW**↓ to turn unit **ON**.
2. The test will default to the last protocol selected.
3. Press and hold **UP ARROW**↑ to turn unit **OFF**.

Placing the Probe

1. Make sure ear canal is clear of debris.
 2. Select a colored silicone eartip and place on probe tube, fully seating it on the probe.
- 
3. Insert probe in the ear. Probe should remain in place without hand-holding.

Changing the Probe Tube

NOTE: Probe tubes do NOT have to be changed for every test, but may need to be replaced if clogged.

1. To remove probe tube, flip eartip up and use to grip tube.





2. Twist slightly and pull tube straight out of probe head.
3. Insert a new probe tube into the probe head until it is fully seated.

Conducting a Test

1. Press **↓CHANGE** to select a different protocol.
2. Press **←CHANGE→** to scroll to desired protocol.
3. Press **UP ARROW**↑ to return to test screen.
4. Press **←L** or **R→** to select test ear and begin test.
5. Probe Check will appear on screen. The scale should be in the green area. If the scale is in the amber area, reposition the probe in the ear.
6. Test will automatically start once probe check is acceptable.
7. When test is finished, Press **←L** or **R→** to test the other ear, or press **↓REVIEW** to review results graph.

NOTE: “Pass” or “Refer” will only be displayed if set in protocol.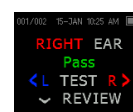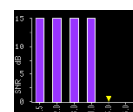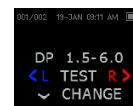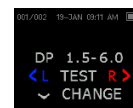

Printing Test Results

NOTE: The ERO•SCAN device has multiple printing options. Setup is required based on the printing method selected.

Option 1: MAICO Sessions

1. Please see the **Operation Manual** for setup instructions.
2. Enter demographics within Patient Information fields.
3. Establish the connection between the ERO•SCAN and the MAICO Sessions software.
4. Once the connection is established the test results are transferred automatically.
5. Select  for direct printout or  for print to PDF.

Option 2: ERO•SCAN Wireless Thermal Printer (optional accessory)

1. Once the test is completed, turn on the printer.
2. Press **UP ARROW**↑ until the printing screen is displayed on the device.
3. Press one arrow next to **←CONNECT→**, for results to be transferred and printing to start.

Option 3: OtoAccess® Database (optional software)

1. Please see the **Instructions for Use** for setup instructions.
2. Select patient from database and enter the MAICO Sessions module.
3. Establish the connection between the ERO•SCAN and the MAICO Sessions software.
4. Once the connection is established the test results are transferred automatically.
5. Select  to save,  for direct printout or  to print to PDF.

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