
GENERAL

To secure correct impedance measurements, it is important to make sure that the probe system is always kept clean. Therefore, please follow the illustrated instructions below on how to remove cerumen from the probe tip's small acoustic and air pressure channels. The instructions apply for all probes of easyTymp and touchTymp.

PREPARATION AND CLEANING


Never clean the probe tip while the tip is still attached to the probe. Unscrew the probe cap by turning it in a counterclockwise direction.

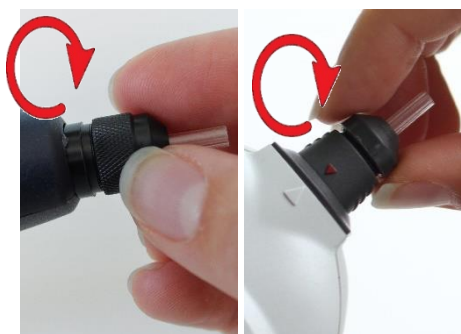touchTymp only: To access and clean the channels, it may be necessary to remove the gasket from inside the probe tip. You can do this using a fine pin. Push the gasket back into place after cleaning.


Proceed in the same way with all probe channels. Use the floss only once.

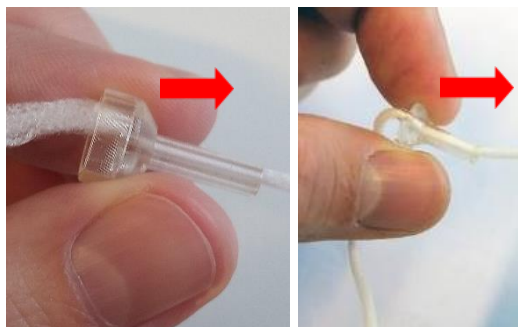Take the plastic probe tip out of the probe.

Insert the blue end of the floss from back to front through one of the probe channels. Pull the floss along its entire length through the channel.

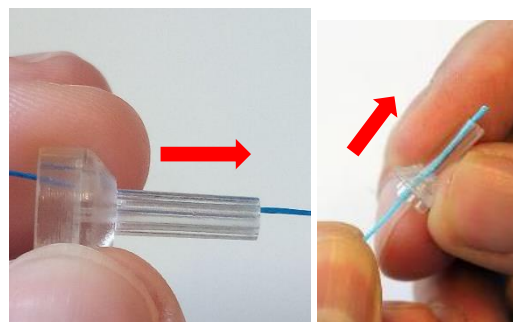
Use a thick or a thin floss depending on the size of the channel.

ASSEMBLING OF THE PROBE TIP

If applicable, place the gasket back on the probe tip and then place the probe tip on the probe. Make sure that the plastic pegs are inserted into the appropriate corresponding cavities.

Screw the probe cap back on the probe. The force of tightening the cap will tighten the screw sufficiently. Never use tools to fix the probe cap!

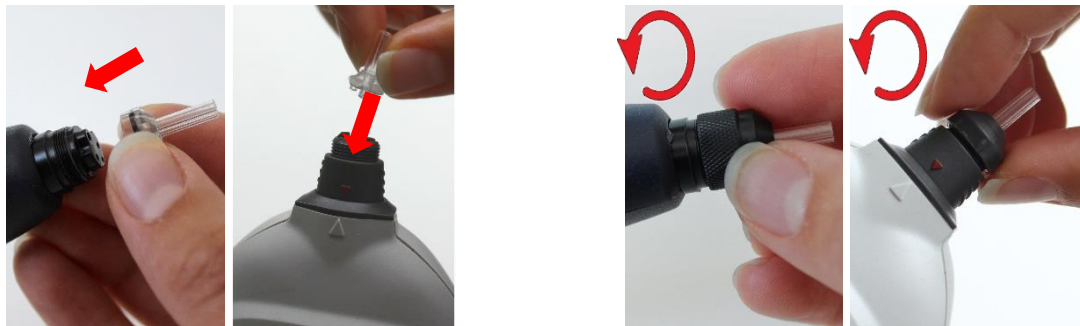ALTERNATIVE

Alternative: Use the cleaning set from the eartip box.

Caution: This procedure destroys the probe.

If the probe tip is clogged or the probe is damaged, only a MAICO authorized distributor can repair the parts. For further information ask your authorized local distributor or MAICO:

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MAICO Diagnostics GmbH