




Getting to Know the Shoulder Box Adapter Kit




Included in the delivery is:

- ✓ Mounting adapter (1)
- ✓ Mounting sleeve (2)
- ✓ 1 Torx screw Ø 2.5x8 (3)
- ✓ Mounting instructions

How to Install the Shoulder Box Adapter Kit on Your Device

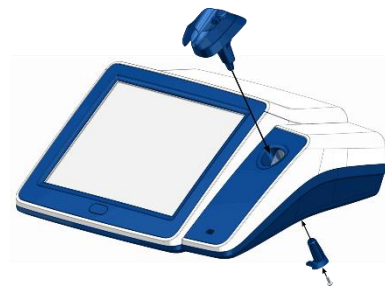

Place the mounting adapter in the insertion on the right-hand side of the device making sure the opening of the mounting adapter is pointing downwards.

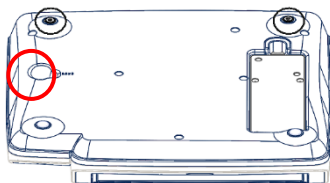

Turn the device and put the mounting sleeve into the hole on the bottom side of the device (red marked area). The anti-twist protection fits into the provided recess.

Place the screw in the hole of the mounting sleeve. Use the Torx key to tighten it with appropriate pressure. The shoulder box adapter is ready for use now.