


1 ABOUT THIS QUICK GUIDE



This Quick Guide is not meant to replace the operation manual. Reading the entire operation manual prior to operating the device is mandatory!
Some of the following functions may vary by the individual settings made.

2 PREPARING THE MEASUREMENT

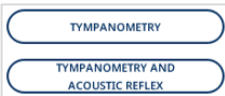


To get started, turn the device on with the power button. Select the test to be performed.



Choose an ear tip of the appropriate size from the ear tip set. Put the ear tip on the probe tip so it meets the ridge of the probe tip.
The placement of the ear tip should cover the entrance and not go into the ear canal.

3 TEST SCREEN OPERATION

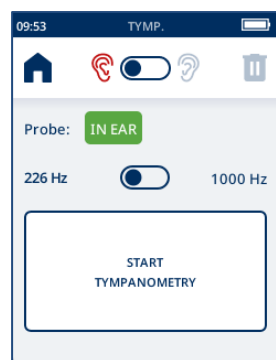

To return to the home screen.



Ear selection: Red for right ear, blue for left ear.




Delete a measurement: From the test screen only the result of the ear selected is deleted. To perform another measurement, any active results must be deleted.
If the icon is inactive, no results are stored for that ear.



Selection of probe tone: This is only visible with devices that have 1 kHz probe tone licensed on device.

4 PERFORMING A TEST

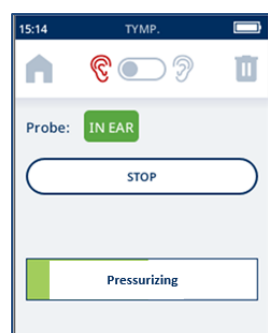

Test start: Once the probe is placed in the ear and a seal achieved, the easyTymp automatically starts the test.

Probe status: This will display **IN EAR**, **BLOCKED** or **READY**. If test does not progress, take out of ear, confirm correct eartip size and placement of probe is directed towards nose and device is horizontal.

Progress bar: Shows the current test stage (Pressurizing or In Progress). When the test is complete, the final result is displayed. Upon completion of test,

- Tympanograms and numerical results display on the screen. Result icons display in the top right corner (✓ = **Normal**, X = **Abnormal**)
- Ipsilateral reflexes graphical representation displays the frequency, level and result symbol (✓ = **Reflex present**, X = **Reflex absent**).

Next Steps:



1. Select the other ear to continue testing.
2. Delete results and retest.
3. Return to home screen to **Print**, **Save** or **Delete** results.



5 RESULT MANAGEMENT


easyTymp offers multiple result management options, which is determined by the configuration purchased.




1. Press  on the **Home** screen to print the results on the thermal printer.
2. Upon completion of printing, press  on the home screen so the device is clear for the next patient.




Transfer results to the PC using MAICO Sessions.

1. Open MAICO Sessions on the PC.
2. Connect the device using the USB cable or by placing it in the cradle.
The results are then transferred automatically.
3. Upon completion of transferring results to MAICO Sessions, press  on the home screen so the device is clear for the next patient.

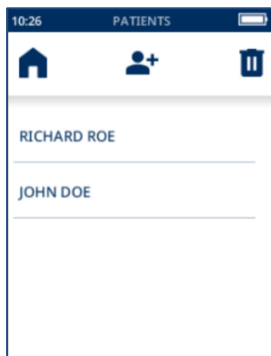

Store the results in the internal memory for

- ✓ later output to the thermal printer or
- ✓ transfer to MAICO Sessions.

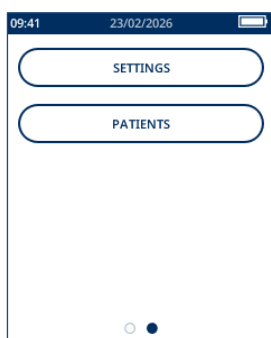

ADDITIONAL FUNCTIONS

Swipe left from the Home screen to access **SETTINGS** and **PATIENTS**.

- ✓ Press **SETTINGS** to open the device settings.
- ✓ Press **PATIENTS** to manage the internal memory.

Refer to the operation manual for details on all options.