

Step-by-Step Guide: Installing the easyTone App on Your Tablet

Step 1: Prepare Your Tablet

- Make sure your tablet is connected to the internet.
- Open the camera or a QR code scanning app.

Step 2: Scan the QR Code

- Use your tablet's camera to scan the QR code provided.



<https://easyTone.app/setup>

- Once scanned, you will be directed to a link to download the app.

Step 3: Download easyTone App

- Tap on the link that appears after scanning the QR code.
- If prompted, tap "Download" to begin downloading the easyTone setup file.

Step 4: Allow Installation

- Depending on your tablet settings, you may need to allow installations from unknown sources.
 - If a pop-up appears, go to **Settings** and enable the option for **Allow from this source**.

Step 5: Install easyTone App

- Once the download is complete, tap on the downloaded file (usually found in the notifications or in the **Downloads** folder).
- Tap **Install/Update** and wait for the installation to be completed.

Step 6: Open easyTone App

- After installation, tap **Open** to launch the app
- Allow access to manage files and select back arrow
- Start using the easyTone!

Optimizing Your Tablet for the Best Experience with easyTone

1. Tablet Requirements

MINIMUM TABLET REQUIREMENTS FOR PERSONAL TABLET

Operating System	Android™
Version	15
Resolution	Full HD (1920 x 1080 px)
Screen Size	10"
Storage	64 GB
RAM	4 GB
Input	USB-C®

2. Accessibility Settings:

To prevent layout issues such as cut-off text or hidden buttons:

- **Font Size:**
Go to Settings > Accessibility > Font size
→ Set to *Default* or *Small*
- **Display Size:**
Go to Settings > Accessibility > Display size
→ Avoid *Large* or *Huge*, as these may disrupt the layout

2. Display Settings:

To avoid interruptions during use:

- **Screen Timeout / Sleep:**
Go to Settings > Display > Screen timeout (or Sleep)
→ Set to **10 minutes** or at least **5 minutes**

3. Optional Accessories for the Best Experience

- **Tablet Stand or Case:**
Use a stand or protective case to improve usability and comfort during operation.
- **Power Supply with USB Splitter:**
If your tablet supports USB splitters, make sure to use a medical-grade power supply. This ensures reliable operation of the easyTone headphone while charging the tablet simultaneously.

QUESTIONS:

For further information,
please contact MAICO at +49 30 +0– 7071 46-42 or tcs@maico.biz.
