

How to update the Platform and Firmware version (using Auto Updater)

Requirement:

SD card

Update files (MA4x_Firmware_Version_15.XX.zip) provided by MAICO TCS or available on Extranet

Precautions:

Do not turn off the device during upgrade process!!!!!!

Do not close the Auto-Programmer using mouse, this will cause the fatal error!!!!

This process should be conducted by Authorized Service partner!!!

Please save the user setting into the SD card before upgrade

Please calibrate the device (Bone conduction), if the device is upgraded from version 14.33.

Devices with firmware before version 15.13 (e.g. version 15.08) must first upgrade to version 15.13 and then to version 15.xx

Procedure:

- (1) Please unzip (MA4x_Firmware_Version_15.XX.zip) using standard zip program, copy the content of Auto updater into SD card. The content of SD card will be as follows (Figure 1).

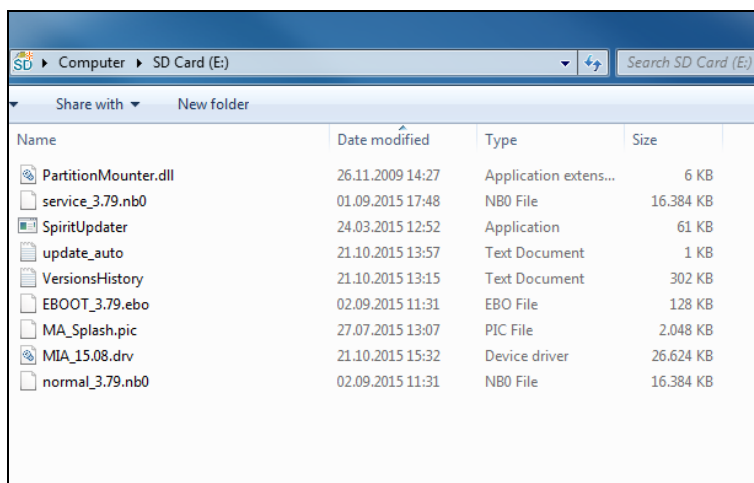


Figure 1

- (2) Now insert the SD card into the device and turn on the device. Auto updater will automatically start the upgrade process as shown in Figure 2.

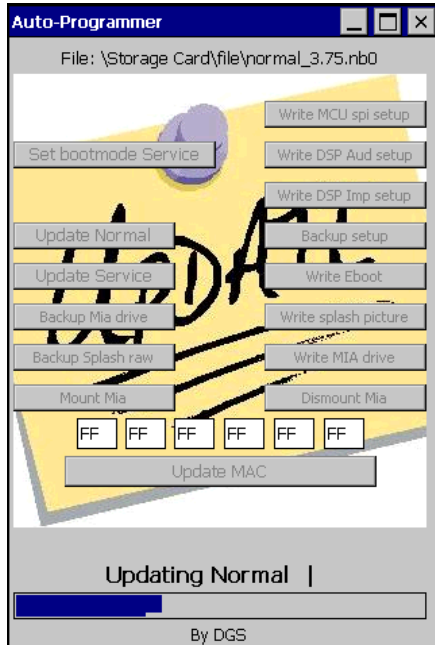


Figure 2

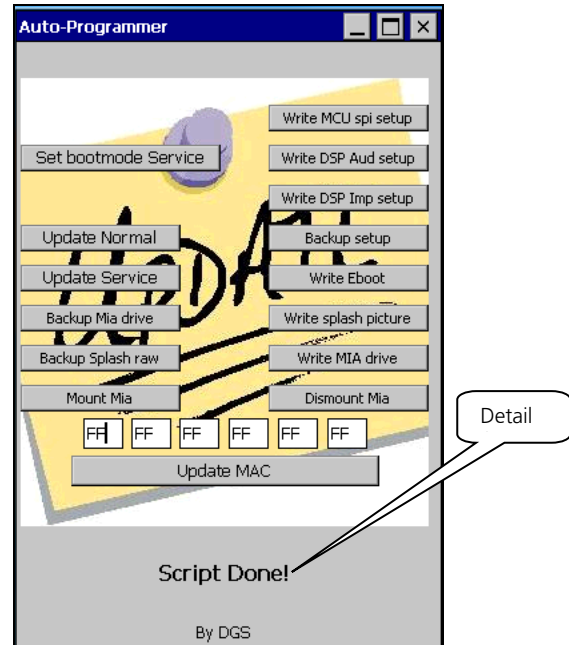


Figure 3

- (3) After successfully completion of upgrade process, "Script Done!" comes up on the screen as shown in Figure 3 (Detail 1). Now remove the SD card and restart the device.
- (4) Please go into Device Information for the verification of Platform and Firmware version.

MAICO Diagnostics GmbH
 Technical Customer Service - TCS
tcs@maico.biz

+ 49 - 30 - 70 71 46 40